GBR  
225 AC

- User and maintenance instruction manual
- Manuel d'utilisation et entretien
- Manual operativo y de mantenimiento

Keep these instructions close to the refrigerator.  
Garder ces instructions près du réfrigérateur.  
Guardar estas instrucciones cerca del refrigerador.

For water quantity, refer page 4.

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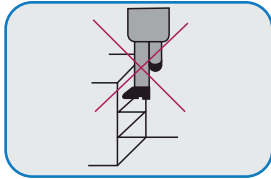
**GBR 225 AC**

**MODEL NUMBER: GBR 225 AC**



Sure Chill® is a patented technology, licensed by The Sure Chill Company

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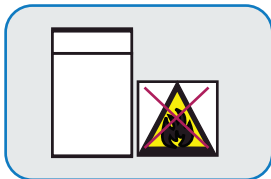
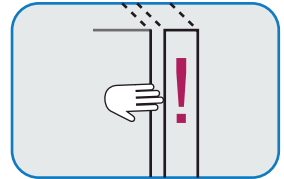
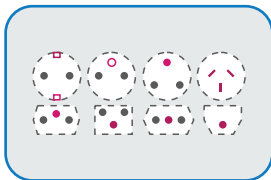
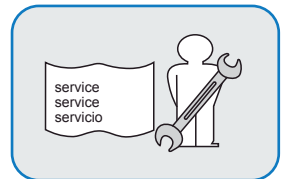
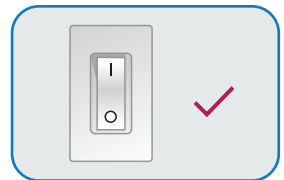
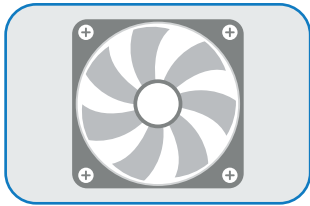
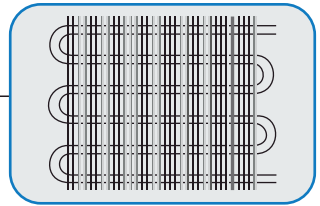
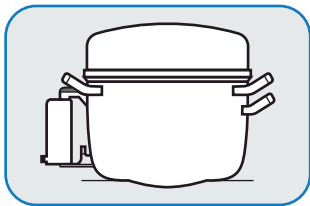
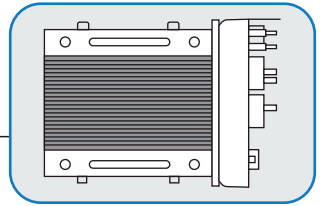
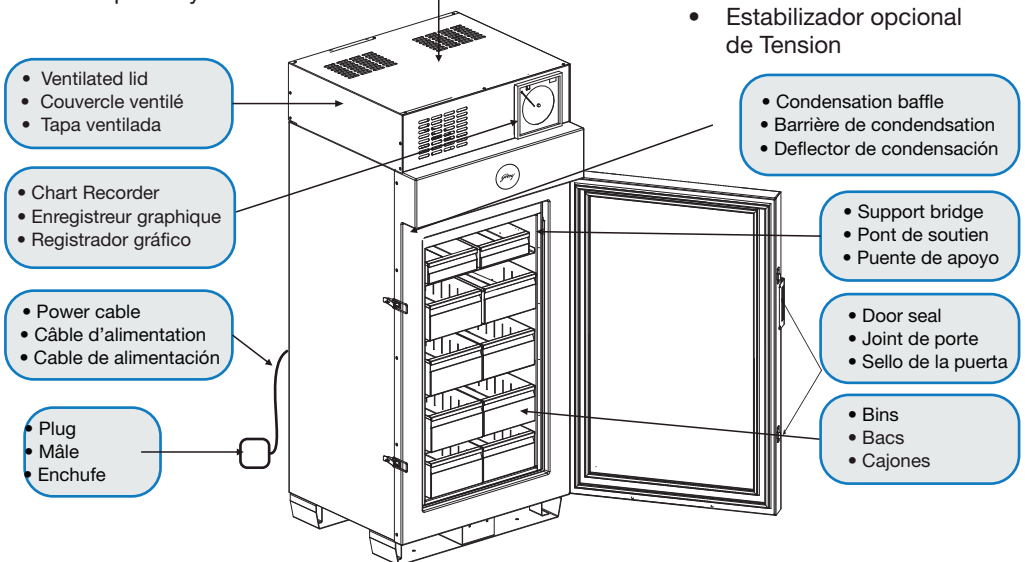
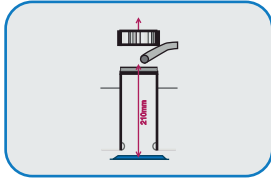
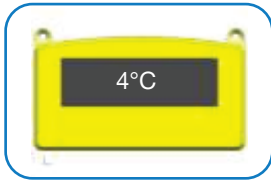
For a full description refer to the latest World Health Organisation publication on the subject: Guidelines for drinking-water quality, fourth edition  
Editors: WHO  
Publication date: 2011  
Languages: English  
ISBN: 978 92 4 154815 1  
[http://www.who.int/water\\_sanitation\\_health/publications/dwq\\_guidelines/en/](http://www.who.int/water_sanitation_health/publications/dwq_guidelines/en/)

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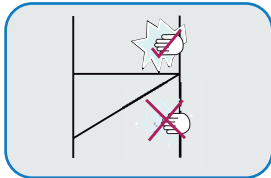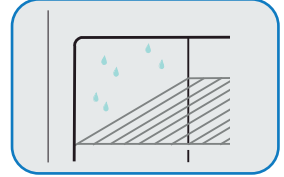
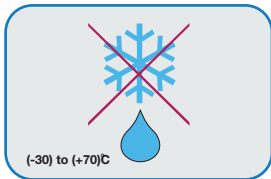
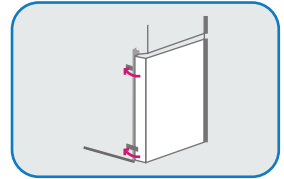
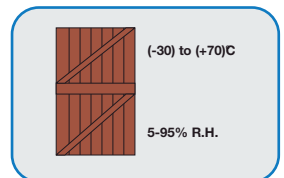
## MAINTENANCE TASKS/ ENTRETIEN/ TAREAS DE MANTENIMIENTO

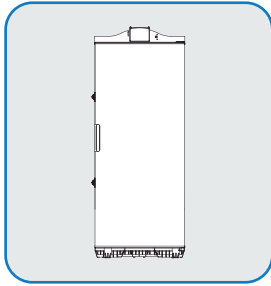**Daily** - confirm compartment temperature is within acceptable range and check all vents are unobstructed

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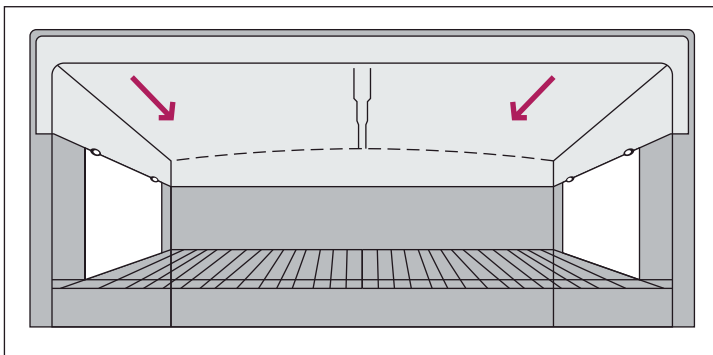
**Weekly** - Clean condensation baffle using soft cloth. Wipe clean and dry door seal and inside surfaces to reduce condensation buildup.

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The product has a built-in intelligent temperature monitoring, malfunction detection and early warning system. A Sensor sounds an audio alarm for 1 min every 15 mins (power supply being available) if the cooling system has failed. In this case, call for technical assistance. Compartment temperature will be maintained between 2-6°C for some additional period post the alarm. In case of alarm, display temperature will have to be monitored frequently. Blood Bags stored in compartment will have to be transferred when safe storage temperatures limit is reached.

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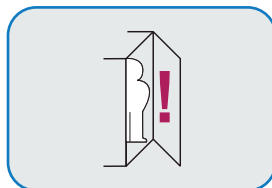
## CUSTOMER SUPPORT/ SOUTIEN CLIENTELE/ SERVICIO AL CLIENTE

Godrej & Boyce Manufacturing Company Limited,  
Pirojshanagar,  
Vikhroli West,  
Mumbai – 400 079  
<http://www.godrejappliances.com/medical>

Board: +91 22 6796 5656 / 5959  
Fax: +91 22 6796 1700 / 1800

E-mail:

Sales Support: [medicalref@godrej.com](mailto:medicalref@godrej.com) / [hello@surechill.com](mailto:hello@surechill.com)  
Service Support: [smartcare@godrej.com](mailto:smartcare@godrej.com)